

OPERATING INSTRUCTION

81365 | VACUUM PUMP COMPACT



The vacuum pump COMPACT from TYROMONT is a product that has been specially developed for operations where every gram counts and perfect functioning must be guaranteed. Due to the use of high-strength plastic, the robust construction and the well thought-out concept, the vacuum pump COMPACT is ideally suited for the tough requirements of rescue services.

Features

Manual vacuum pump made entirely of high-strength plastic. Suction is achieved by raising and lowering the pump piston. Due to the special design of the vacuum pump, suction can be performed by only one person without major effort. The slim cylinder shape makes it easy to reach around with one hand.

The newly designed adapter enables quick and highly efficient suctioning of vacuum mattresses and splints. The cold-resistant silicone hose enables perfect handling even under extreme conditions.

Safety

Heat source or open fire can cause damage to the material or pose a danger to the user. Verify that the pump is in good working order before use.

Storage

Store in a dry place; do not expose to extreme heat or cold sources over an extended period. If the device has not been used for a longer period, the hose should be checked for damage such as cracks.

Cleaning & Disinfection

The vacuum pump can be washed with soap and water. Penetration of liquids at the piston head should be avoided. The vacuum pump can be cleaned with standard disinfectants (wipe disinfection). The use of abrasive cleaners should be avoided as this can cause scratches and damage the material.