

MOUNTING INSTRUCTION

93262 | 93263 GLIDING SKI MODULE TYRAL

The gliding ski modules were developed to facilitate the transport of the mountain stretcher in off-road terrain (snow, forest, and grassy slopes). Thanks to their identical and symmetrical design, incorrect assembly is not possible.

The modules are attached to the stretcher without tools using tube locking pins. Conversion between wheel and ski operation is not necessary – the sliding ski modules can remain attached to the stretcher.

1. Place the TYRAL stretcher upside down on a stable and level surface so that the runners are facing upwards and the stretcher is secure.
2. Position the gliding ski modules at the designated mounting points.
3. The modules are secured by attaching them in front of and behind a longitudinal bar using tube locking pins. Ensure that the pins are fully inserted and locked to prevent them from falling out.
4. Check that all connections are secure.

